<b>DE</b> Läusekamm Gebrauchsanweisung.....	2	<b>IT</b> Pettinina Istruzioni per l'uso .....	36
<b>EN</b> Lice comb Instructions for use.....	11	<b>TR</b> Bit Tarağı Kullanım Kilavuzu .....	44
<b>FR</b> Peigne anti-poux Mode d'emploi .....	19	<b>RU</b> Гребень от вшей Инструкция по эксплуатации.....	52
<b>ES</b> Lendreras Instrucciones de uso .....	27	<b>PL</b> Grzebień na wszy Instrukcja obsługi .....	60

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Read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

## Dear customer,

Thank you for choosing a product from our range. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage, beauty, baby and air.

With kind regards,  
Your Beurer team

---

## Table of contents

1. Included in delivery.....	11	8. Cleaning and maintenance .....	16
2. Signs and symbols.....	12	9. Frequently asked questions .....	16
3. Intended use .....	13	10. Disposal.....	16
4. Warnings and safety notes .....	13	11. Technical specifications .....	17
5. Device description .....	15	12. Warranty/service.....	17
6. Initial use.....	15		
7. Usage.....	15		

---

## 1. Included in delivery

Check that the exterior of the cardboard delivery packaging is intact and make sure that all contents are present. Before use, ensure that there is no visible damage to the device or accessories and that all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Service address.

- 1 x HT 15 lice comb
- 1 x protective cap
- 1 x cleaning brush
- 1 x AA 1.5V battery
- 1 x set of these instructions for use

## 2. Signs and symbols

The following symbols are used in these instructions for use, on the packaging and on the type plate for the device and the accessories:

 <b>WARNING</b>	<b>WARNING</b> denotes danger with medium-level risk, which can lead to death or serious injury if ignored.
 <b>CAUTION</b>	<b>CAUTION</b> denotes danger with low-level risk, which can lead to minor or moderate injury if ignored.
	<b>Product information</b> Note on important information
	<b>Observe the instructions</b> Read the instructions before starting work and/or operating devices or machines
	Application part type B
	Do not dispose of batteries containing harmful substances with household waste
	Manufacturer
	Disposal in accordance with the Waste Electrical and Electronic Equipment EC Directive – WEEE
 PAP	Dispose of packaging in an environmentally friendly manner
<b>LOT</b>	Batch designation
<b>MD</b>	Medical device
<b>IP22</b>	Protected against solid foreign objects 12.5 mm in diameter and larger, and against drops of water when the housing is angled up to 15°
<b>Storage/Transport</b> 	Permissible storage and transport temperature and humidity
<b>Operating</b> 	Permissible operating temperature and humidity
<b>CE</b>	This product satisfies the requirements of the applicable European and national directives

### 3. Intended use

#### Intended use

When it comes into contact with a louse or nit, the lice comb emits a harmless electrical impulse that is imperceptible to humans. When combing the hair, the lice are struck by the electrically charged metal teeth and effectively removed. The lice comb therefore provides a simple, safe and effective method to remove lice.

#### Target group

Use the lice comb to treat hair lice and nits only. The lice comb is intended for domestic/private use only, not for commercial use

#### Indication/clinical benefits

The Beurer HT 15 lice comb allows you to simply and cleanly rid the scalp of lice and unhatched nits (the eggs of the lice). The lice comb does not use any chemicals, meaning it is particularly gentle on the scalp. The device is only intended for combing human head hair, as well as cat and dog fur.

### 4. Warnings and safety notes

#### Contraindications

- Do not use the device on birthmarks or the scalp if the skin is broken.
- Do not use the device on persons with heart conditions, epilepsy or pacemakers.
- People with heart conditions, epilepsy or pacemakers must also not operate the device on other people.

#### General warnings

- Use the device on dry, combed hair only.
- Keep packaging material away from children. Danger of suffocation!
- Do not allow children to play with the device.
- If the lice comb has been used on an animal, it must then never be used on humans.
- Store the device out of the reach of children.
- Children must always be supervised by an adult when using the device.
- After each use, clean the comb thoroughly using the cleaning brush – particularly before using the device on another person. If you would like to share the lice comb with other people, you must first clean the device using a commercially available disinfectant or a disinfectant spray. Please observe the minimum contact times specified by the manufacturer of the disinfectant!
- During use, ensure that the comb does not come into contact with the mouth, eyes or ears.
- People with a sensitive scalp may experience slight tingling or itching during use. This is normal and not harmful to your health.
- Skin contact with the metal teeth during use may result in painful but harmless electrical impulses.
- Do not touch the metal teeth when the device is switched on
- This device can be used by children over the age of 8 and by people with reduced physical, sensory or mental skills or people lacking experience or knowledge, provided that they are supervised or have been instructed on how to use the device safely and are aware of the consequent risks of use.

#### General precautions

- Keep the device away from hot surfaces, and also protect it against direct sunlight.
- When not in use, always place the protective cap provided on the device.
- Operation in the vicinity of strong electromagnetic fields, for example next to a mobile phone, may cause malfunctions.

- Before use, inspect the device and accessories for visible damage. If you have any doubts, do not use the device and contact your retailer or the specified Customer Service address.
- Do not use the device near or in water.
- Never submerge the device in water or other liquids! Make sure that moisture does not get inside the device.
- Protect the device from impacts, moisture, dust, chemicals, marked temperature fluctuations and nearby sources of heat (ovens, heaters).

### Measures for handling batteries

- If your skin or eyes come into contact with battery fluid, rinse the affected areas with water and seek medical assistance.
- Choking hazard! Small children may swallow and choke on batteries. Therefore, store batteries out of the reach of small children.
- Risk of explosion! Do not throw batteries into a fire.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- Do not disassemble, open or crush the batteries.

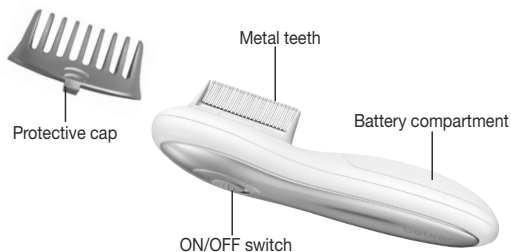
- Observe the plus (+) and minus (-) polarity signs.
- Protect batteries from excessive heat.
- Do not charge or short-circuit batteries.
- If the device is not to be used for a relatively long period, take the batteries out of the battery compartment.
- Use identical or equivalent battery types only.
- Always replace all batteries at the same time.
- Do not use rechargeable batteries.

### Note on electromagnetic compatibility

- The device is suitable for use in all environments listed in these instructions for use, including domestic environments.
- The use of the device may be limited in the presence of electromagnetic disturbances. This could result in issues such as error messages or the failure of the device.
- Avoid using this device directly next to other devices or stacked on top of other devices, as this could lead to faulty operation. If, however, it is necessary to use the device in the manner stated above, this device as well as the other devices must be monitored to ensure they are working properly.
- The use of accessories other than those specified or provided by the manufacturer of this device can lead to an increase in electromagnetic emissions or a decrease in the device's electromagnetic immunity; this can result in faulty operation.
- Failure to comply with the above can impair the performance of the device.

## 5. Device description

## 6. Initial use

### Inserting the battery

1. Slide the battery compartment lid open.
2. Insert the battery included in delivery into the battery compartment as shown inside the battery compartment. When inserting the battery, ensure that the correct polarity (+/-) is observed.
3. Then slide the battery compartment lid closed until you hear and feel it click into place.

## 7. Usage

Only use the lice comb on dry hair. Before use, comb the hair to be treated with a normal comb right to the tips until all tangles are removed.

1. Using your thumb and index finger, press on the notches on the side of the protective cap until it is released. Pull the protective cap up and off.
2. Move the ON/OFF switch to "1". A high-pitched beep sounds, which indicates that the device is switched on and ready for use.
3. Hold the lice comb at an angle of 45° to the scalp. Slowly move the lice comb from the roots to the tips of the hair. Slowly comb through all of the hair carefully and completely.
4. As soon as a louse comes into contact with the metal teeth of the lice comb, the beep audibly changes. If the high-pitched beep does not sound again, clean the metal teeth with the cleaning brush.
5. To end the application, move the ON/OFF switch to "0". The high-pitched beep is switched off.
6. Clean the lice comb after every use as described in the "Cleaning and maintenance" chapter.

### NOTE

The lice comb removes hatched lice as well as unhatched nits (the eggs of the lice). Repeat the application every day for two weeks so that the lice are completely eliminated. In order to prevent lice returning, we recommend repeating the application once or twice a week for a further month. This means that any remaining lice can be removed.

## 8. Cleaning and maintenance

### CAUTION

- Remove the protective cap for better cleaning of the metal teeth. Carefully clean the metal teeth after every use using the cleaning brush provided. Check the lice comb for any remnants between the metal teeth.
- Clean the device (apart from the metal teeth) after use with a soft, slightly damp cloth. If it is very dirty, you can also moisten the cloth with a mild soapy solution. Do not use any abrasive cleaning products.
- Ensure that no water enters the device. If this should happen, remove the batteries and only use the device again when it is completely dry.

## 9. Frequently asked questions

### Who can be affected by head lice?

Anyone can be affected by head lice, not only children. Having a case of lice is also not caused by a lack of hygiene.

### How do you get head lice?

Lice cannot fly or jump; they scuttle and crawl. Lice are transferred by direct contact with a person that already has lice (for example when two heads come into contact).

### Where do head lice come from?

School is usually not the source of transfer. Lice are found much more often in communities outside of school.

### Who should be checked and how?

If you have been informed of lice in your community, check all members of your family at least once a week and inform your friends.

### Why is it important to ascertain whether lice are present immediately?

If you find out promptly that lice are present, it is easier to remove them as their numbers will be lower. There is also a lower probability that the lice will be transferred to other family members or acquaintances.

## 10. Disposal

For environmental reasons, do not dispose of the device in the household waste at the end of its useful life. Dispose of the device at a suitable local collection or recycling point in your country. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment).

If you have any questions, please contact the local authorities responsible for waste disposal. Batteries must not be disposed of in the household waste. They may contain poisonous heavy metals and are subject to special refuse treatment. The codes below are printed on batteries containing harmful substances:

Pb = battery contains lead,

Cd = battery contains cadmium,

Hg = battery contains mercury.

## 11. Technical specifications

Model:	HT 15
Battery:	1 x 1.5V AA batteries
Weight:	Approx. 56 g (without batteries)
Dimensions:	158 x 37 x 50 mm
Permissible storage and transport temperature and humidity:	-10°C to 50°C, 10% to 50% relative humidity
Permissible operating temperature and humidity:	10°C to 40°C, 30% to 85% relative humidity

The batch number is located on the device or in the battery compartment.  
Subject to technical changes.

The device complies with the Regulation (EU) 2017/745 of the European Parliament and of the Council on medical devices, as well as the respective national provisions (in accordance with IEC 61000-4-2, IEC 61000-4-3) and is subject to particular precautions with regard to electromagnetic compatibility. The device is suitable for use in all environments listed in these instructions for use, including domestic environments.

### Notes on electromagnetic compatibility

- The device is suitable for use in all environments listed in these instructions for use, including domestic environments.
- The use of the device may be limited in the presence of electromagnetic disturbances. This could result in issues such as error messages or the failure of the display/device.
- Avoid using this device directly next to other devices or stacked on top of other devices, as this could lead to faulty operation. If, however, it is necessary to use the device in the manner stated, this device as well as the other devices must be monitored to ensure they are working properly.
- The use of accessories other than those specified or provided by the manufacturer of this device can lead to an increase in electromagnetic emissions or a decrease in the device's electromagnetic immunity; this can result in faulty operation.
- Failure to comply with the above can impair the performance of the device.

## 12. Warranty/service

Beurer GmbH, Söflinger Straße 218, 89077 Ulm, Germany (hereinafter referred to as "Beurer") provides a warranty for this product, subject to the requirements below and to the extent described as follows.

**The warranty conditions below shall not affect the seller's statutory warranty obligations which ensue from the sales agreement with the buyer.**

**The warranty shall apply without prejudice to any mandatory statutory provisions on liability.**

Beurer guarantees the perfect functionality and completeness of this product.

The worldwide warranty period is 3 years, commencing from the purchase of the new, unused product from the seller.

The warranty only applies to products purchased by the buyer as a consumer and used exclusively for personal purposes in the context of domestic use.

German law shall apply.

During the warranty period, should this product prove to be incomplete or defective in functionality in accordance with the following provisions, Beurer shall carry out a repair or a replacement delivery free of charge, in accordance with these warranty conditions.

**If the buyer wishes to make a warranty claim, they should approach their local retailer in the first instance: see the attached “International Service” list of service addresses.**

The buyer will then receive further information about the processing of the warranty claim, e.g. where they can send the product and what documentation is required.

A warranty claim shall only be considered if the buyer can provide Beurer, or an authorised Beurer partner, with

- a copy of the invoice/purchase receipt, and
- the original product.

The following are explicitly excluded from this warranty:

- deterioration due to normal use or consumption of the product;
- accessories supplied with this product which are worn out or used up through proper use (e.g. batteries, rechargeable batteries, cuffs, seals, electrodes, light sources, attachments and nebuliser accessories);
- products that are used, cleaned, stored or maintained improperly and/or contrary to the provisions of the instructions for use, as well as products that have been opened, repaired or modified by the buyer or by a service centre not authorised by Beurer;
- damage that arises during transport between manufacturer and customer, or between service centre and customer;
- products purchased as seconds or as used goods;
- consequential damage arising from a fault in this product (however, in this case, claims may exist arising from product liability or other compulsory statutory liability provisions).

Repairs or an exchange in full do not extend the warranty period under any circumstances.

For users/patients in the European Union and identical regulation systems (EU Medical Device Regulation (MDR) 2017/745), the following applies: If during or through use of the product a major incident occurs, notify the manufacturer and/or their representative of this as well as the respective national authority of the member state in which the user/patient is located.

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