beurer
_____medical

FT 90

**READ THIS MANUAL
COMPLETELY AND
CAREFULLY BEFORE
USING THIS PRODUCT**

Keep this manual in a safe location
for future reference

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ENGLISH Non-contact forehead thermometer
Instruction manual 2

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Questions or comments?
Call toll free 1-800-536-0366 or contact
info@beurer.com

CE 0483

SAVE THESE INSTRUCTIONS FOR FUTURE REFERENCE

If the instruction manual is damaged or if you no longer have the instruction manual in your possession, please contact Beurer customer service. Please see warranty for service contact.

Dear customer,

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, gentle therapy, blood pressure/diagnosis, weight, massage and air.

Please read these instructions for use carefully and observe the information they contain.

With kind regards,
Your Beurer Team

Indication for use

The non-contact forehead thermometer, Model FT 90, is an infrared thermometer intended for the periodic measurement of human body temperature for persons of all ages. Additionally it can measure the room and object temperature in seconds.

1. Important safety notes

Signs and symbols

The following signs appear in the Safety Section (page 2, 3, 4, 5, 6) and in this manual on pages 13, 14, 18, 19, 20.

READ THIS ENTIRE MANUAL INCLUDING THE SAFETY SECTION AND ALL INSTRUCTIONS AND WARNINGS COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT. FOLLOW ALL SAFETY INSTRUCTIONS AND WARNINGS TO AVOID HAZARDOUS SITUATIONS AND TO MAKE CORRECT USE OF THIS PRODUCT.

	WARNING	WARNING indicates a hazardous situation which, if not avoided, could result in death or serious injury.
	CAUTION	CAUTION indicates a hazardous situation which, if not avoided, may result in minor or moderate injury.
	NOTICE	NOTICE addresses practices not related to personal injury, such as product and/or property damage.
		Type BF applied part
		Follow instructions for use
		Operating conditions
		Storage conditions
		Transport conditions
	SN	Serial number
	0483	The CE labelling certifies that the product complies with the essential requirements of Directive 93/42/EEC on medical products.

WARNING:

- Prolonged high fever requires medical attention-especially in young children.
- The thermometer needs to be in the room in which the measurement is taken for at least 30 minutes before use.

**WARNING:**

- The FT 90 thermometer is only designed for the measuring point on the human body stated in the instructions for use.
- The device is only intended for the purpose stated in these instructions for use.
- Children must not be allowed to use the device. Medical products are no toys.
- The device has been designed for practical use, but is not a substitute for a visit to the doctor.
- Should you have any questions about using the device, please contact either your retailer or Customer Services.
- Packaging materials are a deadly hazard for children and can cause suffocation. Remove all packaging materials immediately and keep them away from children at all times.
- Swallowing batteries and/or battery fluid can be extremely dangerous. Keep the battery out of the reach of children. Should a child, or any other person swallow a battery and/or battery fluid, please call 911 immediately.
- Should battery fluid leak and come into contact with your eyes or skin, immediately rinse with plenty of clean water. Please call 911 immediately.
- Batteries should not be charged or reactivated by any other means.
- Batteries should not be taken apart, thrown in the fire or short circuited.

**CAUTION:**

- Check before each use that the measuring sensor is intact. If it is damaged, please contact your retailer or the service address.
- This unit is intended for home use only. It is not intended for professional use in hospitals or other medical facilities. This unit does not support the standards required for professional use.
- If a battery has leaked do not touch the battery fluid. Avoid skin contact (e.g. put on protective gloves) and clean the battery compartment with a dry cloth.
- Do not disassemble this unit.

NOTICE:

- Portable and mobile HF communication systems may interfere with this device. For more details, please contact our customer service at the address indicated or see the enclosed information sheet.
- Do not subject the device to mechanical impacts.
- When not in use, store the thermometer in the provided storage case.
- Do not expose the device to liquids. The device is not waterproof. Avoid all direct contact with water or other liquids.
- Do not expose the device to direct sunlight.
- Clean the measuring sensor before each use.
- Do not touch the measuring sensor with fingers.
- Do not use any aggressive solvents, cleaning agents, detergents or any other strong chemicals to clean the device.
- Always use the size and type of batteries indicated.
- Never use rechargeable batteries. This may damage the unit.
- Replace weak batteries before they discharge completely. When “” in the display occurs please change the batteries.
- Leaking batteries may damage the device. If you do not intend to use the thermometer for longer periods, remove the batteries from the battery compartment before placing the device in storage.
- Batteries can contain toxins that are harmful to the environment. Always dispose of batteries in accordance with applicable local regulations. Do not dispose of batteries with normal household waste.
- Observe the local regulations for material disposal. Dispose of the device in accordance with local regulations.
- Do not use mobile telephones, microwaves, and other devices which generate strong electrical or electromagnetic fields near the unit. This may result in incorrect operation of the unit.
- Do not press the buttons with excessive force or with any pointed objects.

NOTICE:

- Never attempt to repair the instrument or adjust it yourself. This may damage the unit and impair the functions. If you need to have the unit repaired, please contact our customer service. Please see warranty for service contact. Before submitting any complaint, first check the batteries and replace them if necessary.
- Changes or modifications to the device will nullify the user warranty.

Indication for use	2
1. Important safety notes	2
2. Included in delivery	6
3. Information about this thermometer	7
4. Device description	8
5. Initial use	9
6. Switching on and setting the thermometer	10
7. Forehead measurement	13
8. Measuring object temperature/room temperature	16
9. Error messages	17
10. Batteries	18
11. Cleaning the device	20
12. Storing the device	21
13. Disposing of the device	21
14. Technical Data	21
15. Warranty	23

2. Included in delivery

- Non-contact forehead thermometer
- 2 x AAA (LR03) batteries
- Storage box
- Instructions for use
- Quick Start Guide

3. Information about this thermometer

The temperature measurement varies depending on the part of the body where the measurement is taken. In a healthy person, the variance can be between 0.4°F and 1.8°F (0.2°C and 1°C) in different parts of the body.

The result of FT 90 is shown as oral temperature.

Beurer tip

- Temperatures measured with different thermometers should never be compared with one another.
 - Tell your doctor what type of thermometer you used to take your temperature and on what part of the body. Also bear this in mind if you are diagnosing yourself.
-

Holding the thermometer for too long in the hand before taking a measurement can cause the device to warm up. This means the measurement could be incorrect.

Influences on body temperature

- A person's individual metabolism
- Age

Body temperature is higher in babies and toddlers than in adults. Greater temperature fluctuations occur faster and more often in children. Normal body temperature decreases with age.

- Clothing
- Outside temperature
- Time of day

Body temperature is lower in the morning and increases throughout the day towards evening.

- Activities

Physical and, to a lesser extent, mental activities increase body temperature.


Beurer tip

Taking the body temperature provides a current measurement of a person's temperature. If you are uncertain about interpreting the results or if the values are abnormal (e.g. fever), please consult your doctor. This also applies in the case of slight temperature changes if there are other symptoms of illness such as agitation, severe sweating, flushed skin, fast pulse rate, tendency to collapse, etc.

4. Device description


- 1 Display
- 2 **MODE/MEM** button
- 3 On/Off button **⓪**
- 4 **LIGHT/SET** button
- 5 **Start-measurement** button
- 6 Measuring sensor
- 7 Battery compartment lock
- 8 Battery compartment lid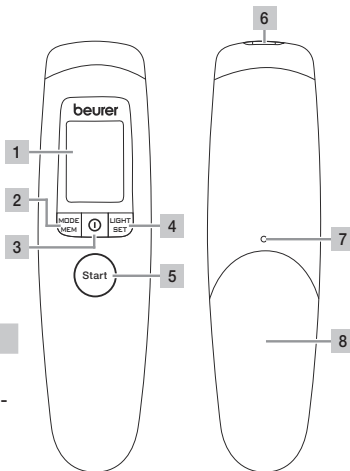

Buttons	Functions
⓪	Turns the device on and off.
Start	Starts the temperature measurement.
MODE/MEM (short)	Sets the measurement mode.
MODE/MEM (5 sec.)	Displays stored measurements.
LIGHT/SET (short)	Manual switch on of the illuminated display.
LIGHT/SET (long)	Sets the basic functions.

4.1 Display description

- 1 Object temperature mode
- 2 Room temperature mode
- 3 Forehead temperature mode
- 4 Acoustic signal symbol
- 5 Year
- 6 Memory function
- 7 Date
- 8 Battery level
- 9 Temperature/memory space number display
- 10 Fahrenheit/Celsius measurement unit
- 11 Measurement result: "fever"
- 12 Measurement result: "no fever"
- 13 Illuminated display symbol
- 14 Year/date/time display

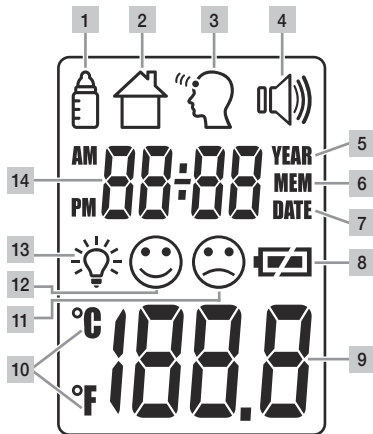5. Initial use

If provided, pull out the battery insulating strip at the battery compartment or remove the protective film from the battery and insert the battery with the correct polarity.

▷ 10. Batteries

After a brief self-test and two short beeps the thermometer is ready for forehead measurement.

6. Switching on and setting the thermometer

The measuring sensor should be cleaned before and after each use, to eliminate dirt and fingerprints and ensure accuracy of your readings.

Briefly press the On/Off button **ⓘ**.

After a brief self-test and two short beeps the thermometer is ready for forehead measurement.

The device always starts up in forehead temperature mode .

6.1 Setting the basic functions

This menu allows you to set the following functions individually, one after another.

Time format

- With the thermometer switched on, press and hold the **LIGHT/SET** button for 5 seconds.
The time format option flashes on the display (fig. 1).
- Use the **MODE/MEM** button to set your preferred time format and confirm with the **LIGHT/SET** button.

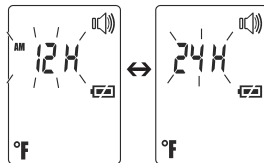
Fig. 1

The year flashes in the display (fig. 2).

- Use the **MODE/MEM** button to set the year and confirm with the **LIGHT/SET** button.

Month/day flashes in the display (fig. 3).

- Use the **MODE/MEM** button to set the month and day, and confirm with the **LIGHT/SET** button.

- ❶ In the 24h format, the date is displayed as day/month. In the 12h format, it is displayed as month/day.

Fig. 2

Fig. 3

The time flashes in the display (fig. 4).

- Use the **MODE/MEM** button to set the time and confirm with the **LIGHT/SET** button.

- ❶ In the 12h format, the time is displayed as AM/PM.

Fig. 4

The temperature measurement unit flashes in the display (fig. 5).

You can set the device to display the temperature in degrees Fahrenheit (°F) or degrees Celsius (°C).

- To display the temperature in Celsius, select °C using the **MODE/MEM** button and confirm with the **LIGHT/SET** button.
- To display the temperature in Fahrenheit, select °F using the **MODE/MEM** button and confirm with the **LIGHT/SET** button.

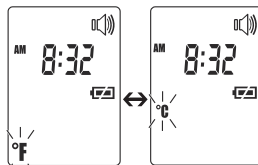
Fig. 5

The acoustic signal symbol flashes in the display (fig. 6). You can activate/deactivate the acoustic signals (device activation, measurement in progress, measurement completion).

- To activate the acoustic signals, select **ON** using the **MODE/MEM** button and confirm with the **LIGHT/SET** button.
- To deactivate the acoustic signals, select **OFF** using the **MODE/MEM** button and confirm with the **LIGHT/SET** button.

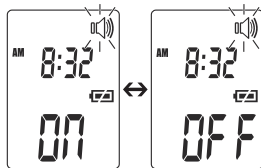
Fig. 6

The light symbol flashes in the display (fig. 7).

You can activate or deactivate the automatic illuminated display (appears following forehead temperature measurement).

- To activate the automatic illuminated display, select **ON** using the **MODE/MEM** button and confirm with the **LIGHT/SET** button.
- To deactivate the automatic illuminated display, select **OFF** using the **MODE/MEM** button and confirm with the **LIGHT/SET** button.

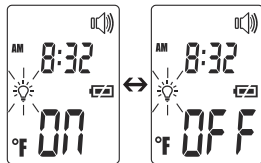
Fig. 7

You can also activate the illuminated display manually by briefly pressing the **LIGHT/SET** button.

The display lights up for 5 seconds.

- ⓘ The automatic and manual illuminated display cannot be used while measurement is in progress.

7. Forehead measurement

WARNING:

- Prolonged high fever requires medical attention-especially in young children.
- Only use the device once you have read and understood these instructions for use.
- The thermometer needs to be in the room in which the measurement is taken for at least 30 minutes before use.
- The FT 90 thermometer is only designed for the measuring point on the human body stated in the instructions for use.
- The device is only intended for the purpose stated in these instructions for use.
- Children must not be allowed to use the device. Medical products are not toys.
- The device has been designed for practical use, but is not a substitute for a visit to the doctor.
- Should you have any questions about using the device, please contact either your retailer or Customer Services.
- Packaging materials are a deadly hazard for children and can cause suffocation. Remove all packaging materials immediately and keep them away from children at all times.

CAUTION:

- Check before each use that the measuring sensor is intact. If it is damaged, please contact your retailer or the service address.
- This unit is intended for home use only. It is not intended for professional use in hospitals or other medical facilities. This unit does not support the standards required for professional use.
- Do not disassemble this unit.

NOTICE:

- Do not use mobile telephones, microwaves, and other devices which generate strong electrical or electromagnetic fields near the unit. This may result in incorrect operation of the unit.
- Do not subject the device to mechanical impacts.
- Portable and mobile HF communication systems may interfere with this device. For more details, please contact our customer service at the address indicated or see the enclosed information sheet.
- Do not press the buttons with excessive force or with any pointed objects.
- Never attempt to repair the instrument or adjust it yourself. This may damage the unit and impair the functions. If you need to have the unit repaired, please contact our customer service. Please see warranty for service contact. Before submitting any complaint, first check the batteries and replace them if necessary.
- Changes or modifications to the device will nullify the user warranty.

Briefly press the On/Off button .

After a brief self-test and two short beeps the device is ready for forehead temperature measurement. The device is in forehead temperature mode. This is indicated by the  symbol.

**Beurer tip**

Remember that

- physical activity, increased perspiration on the forehead, taking vasoconstrictive medication and skin irritations can distort the result.
- the forehead must be free from perspiration and cosmetics.
- before taking a reading, please remain indoors at room temperature for 5 minutes avoiding exercise or taking a bath for 30 min prior reading.
- in case of doubt, please measure several times and take the highest measurement value.

Hold the thermometer 0.78" – 1.18" from the indicated point on the middle of the forehead (Fig.8) and press the **Start-measurement** button. The end of measurement is signaled by a long beep. The measuring time is usually less than 1 second. Release the **Start-measurement** button. You can now read the measurement value.

Note: Only one single measurement is taken at a single spot in the middle of the forehead.

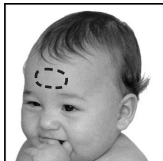
Fig. 8

In addition to the temperature, the fever ☹️ or no fever 😊 symbols also appears in the display:

The no fever symbol 😊 indicates that the body temperature is within normal range; the fever symbol ☹️ indicates a measurement of over 100.4°F, i.e. temperature alarm.

The device is ready to take another measurement when two short beeps sound and the forehead symbol 🧠 stops flashing.

The measurement is automatically saved with the date/time and the "fever" ☹️ / "no fever" 😊 classification.

Note:

- It is not necessary to find an artery as the FT 90 (oral measurement equal) reads oral temperature based on the skin and ambient temperature of one point in the middle of the forehead.
- There is no need to scan across the skin.
- Holding the thermometer in one place too long may cause a higher ambient temperature reading which could make the body temperature reading lower than usual.

7.1 Displaying saved measurements

The device only stores measurements in forehead temperature mode . The device automatically stores the values from the last 60 measurements.

When 60 storage places are exceeded the oldest value is deleted.

The memory can be called up as follows:

- With the thermometer switched on, press and hold the **MODE/MEM** button for 5 seconds.
The most recent measurement is displayed.
- With each subsequent pressing of the **MODE/MEM** button, first the storage space number is displayed followed by the measurement upon release.
- In the upper line, the time and date are displayed alternately.

8. Measuring object temperature /room temperature

Object temperature mode

If you want to measure an object temperature with this thermometer, switch to object temperature mode.

- With the thermometer switched on, briefly press the **MODE/MEM** button.
The device switches to the object temperature mode .
- Hold the thermometer 0.78"-1.18" from the measuring point. Briefly press the **Start-measurement** button and read the temperature in the display (fig. 9).

Measurements taken in object temperature mode are not stored.

Fig. 9

If you want to use the thermometer to measure the room temperature you need to switch to room temperature mode.

- With the thermometer switched on, briefly press the **MODE/MEM** button twice.

The device switches to the room temperature mode .

- *The room temperature is immediately displayed (fig. 10).*

Measurements taken in room temperature mode are not stored.

Fig. 10

9. Error messages

Error message	Problem	Solution
	Measurement during self-test, device not yet ready for measurement.	Wait until the forehead symbol stops flashing.
	Large variation in room temperature.	Store the device for at least 30 minutes in a room with room temperature (50–104 °F / 10–40 °C).
	Room temperature below <50 °F, >104 °F (<10 °C or over 40 °C).	Room temperature must be between 50 °F and 104 °F (10 °C and 40 °C).
	Faultless functionality cannot be guaranteed.	Remove the battery for approx. 1 minute and then reinsert. In the event of a repeated error message, contact your retailer or Customer Services.

Error message	Problem	Solution
Hi	(1) Forehead temperature mode: The temperature recorded is higher than 108 °F (42.2 °C). (2) Object temperature mode: The temperature recorded is higher than 176 °F (80 °C).	Operate the thermometer only between the specified temperature ranges. In the event of a repeated error message, contact your retailer or Customer Services.
Lo	(1) Forehead temperature mode: The temperature recorded is lower than 93.2 °F (34 °C). (2) Object temperature mode: The temperature recorded is lower than -7.6 °F (-22 °C).	Operate the thermometer only between the specified temperature ranges. In the event of a repeated error message, contact your retailer or Customer Services.
	Self-test unsuccessful.	Replace the batteries.
	The batteries are empty.	Replace the batteries.

10. Batteries

WARNING:

- Swallowing batteries and/or battery fluid can be extremely dangerous. Keep the battery and scale out of the reach of children. Should a child, or any other person swallow a battery and/or battery fluid, please call 911 immediately.
- Should battery fluid leak and come into contact with your eyes or skin, immediately rinse with plenty of clean water. Please call 911 immediately.
- Batteries should not be charged or reactivated by any other means.
- Batteries should not be taken apart, thrown in the fire or short circuited.

CAUTION:

- If a battery has leaked do not touch the battery fluid. Avoid skin contact (e.g. put on protective gloves) and clean the battery compartment with a dry cloth.

NOTICE:

- Always use the size and type of batteries indicated.
- Never use rechargeable batteries. This may damage the unit.
- Replace weak batteries before they discharge completely. When “” appears in the display please change the batteries.
- Leaking batteries may damage the device.
If you do not intend to use the thermometer for longer periods, remove the batteries from the battery compartment before placing the device in storage.
- Batteries can contain toxins that are harmful to the environment. Always dispose of batteries in accordance with applicable local regulations. Do not dispose of batteries with normal household waste.


The device requires two AAA (LR03) batteries.

- Open the battery compartment.
Use a pointed object to press down on the battery compartment lock and at the same time slide the battery compartment downwards.
- Take the used batteries out of the battery compartment.
- Insert new batteries.

Make sure that the batteries are inserted the right way round.

- Close the battery compartment.

Used batteries should not be disposed of in normal household waste. You are legally required to dispose of the batteries. Dispose of them via your electronics retailer or your local recycling point.

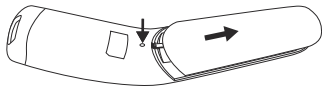Note: The codes below are printed on batteries containing harmful substances: Pb = battery contains lead, Cd = battery contains cadmium, Hg = battery contains mercury. The batteries in this device do not contain any pollutants.

11. Cleaning the device

NOTICE:

- Do not subject the device to mechanical impacts.
- When not in use, store the thermometer in the provided storage case.
- Do not expose the device to liquids. The device is not waterproof. Avoid all direct contact with water or other liquids.
- Do not expose the device to direct sunlight.
- Clean the measuring sensor before each use.
- Do not touch the measuring sensor with fingers.

- The measuring sensor is the most sensitive part of the thermometer. Take great care of the measuring sensor when cleaning the device.
 - Do not use any harsh cleaning products.
 - Always observe all safety notes for user and device.
Safety notes ▷ Page 2 – 6.
-

Clean the measuring sensor after each use. Use a clean cloth or cotton bud that can be moistened with disinfectant or 70 % alcohol.
To clean the entire device, please use a soft cloth slightly moistened with a mild soapy solution. Under no circumstances may liquid enter the device.
Do not use the device again until it is completely dry.

12. Storing the device

The device must not be stored or used at an excessively high or low temperature or humidity (see technical specifications), in sunlight, in association with an electrical current or in dusty locations. Otherwise inaccuracies can occur.

If prolonged storage is planned, you should remove the batteries.

When not in use, store the thermometer in the provided storage case.

13. Disposing of the device

Observe the local regulations for material disposal. Dispose the device in accordance with local regulations.

14. Technical Data

If the device is used other than in accordance with the specifications, perfect functioning cannot be guaranteed!

We reserve the right to make technical changes to improve and develop the product.

Measurement method	Non-contact infrared measurement
Basic functions	Forehead temperature measurement Object temperature measurement Room temperature measurement
Measurement units	Fahrenheit (°F) and Celsius (°C)
Operating conditions	50 °F to 104 °F (10 °C to 40 °C) with a relative humidity of ≤95 %
Storage conditions	-4 °F to 122 °F (-20 °C to 50 °C) with a relative humidity of ≤85 %
Transport conditions	<158 °F (70 °C) with a relative humidity of ≤95 %
Measurement distance	0.78" to 1.18" from the measuring point

Measurement range and accuracy of forehead temperature measurement	Forehead temperature measurement 93.2 °F to 108 °F (34 °C to 42.2 °C) Measurement accuracy 96.8 °F to 102 °F: ± 0.4 °F (36 °C to 39 °C: ± 0.2 °C) In the remaining measurement range ± 0.5 °F (± 0.3 °C)
Clinical repeat precision	0.41 °F (0.23 °C)
Measurement range and accuracy Object temperature measurement	Object temperature measurement -7.6 °F to 176 °F (-22 °C to 80 °C) Measurement accuracy $\pm 4\%$ or ± 4 °F (± 2 °C)
Memory function Body temperature measurement	Automatically stores the last 60 measurements.
Acoustic signal	With device activation, during measurement, when measurement is complete (can be set).
Display	LCD display
Energy saving functions	Device automatically switches off after 1 minute.
Dimensions Width x depth x height	aprox. 1.87" x 1.14" x 7.4" (aprox. 47,6 mm x 29,0 mm x 188,0 mm)
Weight	2.8 oz (82 g) (without batteries)
2 x AAA (LR03) batteries	Batteries last for approx. 3000 measurements Activated functions such as acoustic signal or illuminated display also reduce the battery life.

These specifications are subject to change without notice for purpose of improvement.

- This unit is in line with the International Standard IEC 60601-1-2 and is subject to particular precautions with regard to electromagnetic compatibility (EMC). Please note that portable and mobile HF communication systems may interfere with this instrument. For more details, please contact customer service.
- This device meets the requirements of the EU Directive 93/42/EC concerning medical devices, as well as those of the Medizinproduktegesetz (German Medical

Devices Act), the ASTM E 1965-98 and the European Standard EN60601-1-2, and is subject to particular precautions with regard to electromagnetic compatibility. Please note that portable and mobile HF communication systems may interfere with this unit. More details can be requested from the stated Customer Service address or found at the end of the instructions for use.

15. Warranty

Limited Lifetime Warranty For Original Purchaser

Your Beurer Thermometer, Model FT 90, excluding the battery, is warranted to be free from defects in materials and workmanship for the life of the product under normal conditions of intended use and service. This warranty extends only to the original retail purchaser and does not extend to retailers or subsequent owners.

We will, at our option, repair or replace the Beurer Thermometer, Model FT 90, without additional charge, for any part or parts covered by these written warranties. No refunds will be given. Repair or replacement is our only responsibility and your only remedy under this written warranty. If replacement parts for defective materials are not available, Beurer reserves the right to make product substitutions in lieu of repair or replacement.

For warranty service contact our customer service department at **1-800-536-0366** or at **info@beurer.com** to provide a description of the problem. If the problem is deemed to be within the scope of the limited lifetime warranty, you will be asked to mail the product at your costs in its original package with proof of purchase, your name, address and phone number. If the problem is not deemed to be within the scope of the limited lifetime warranty, we will provide a quotation for repair respectively replacement and return shipping fee.

This warranty does not cover damage caused by misuse or abuse; accident; the attachment of unauthorized accessory; alteration to the product; improper installation; misapplication; lack of reasonable care with respect to the product; unauthorized repairs or modifications; improper use of electrical/power supply; old worn battery; normal wear; loss of power; dropped product; malfunction or damage of an operating part as a result of failure to comply with instructions for use or to provide manufacturer's recommended maintenance; transit damage; theft; neglect; vandalism; or environmental conditions; loss of use

during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of Beurer. This warranty is void if the product is ever used in a commercial or business environment. The maximum liability of Beurer under this warranty is limited to the purchase price actually paid by the customer for the product covered by the warranty, as confirmed by proof of purchase, regardless of the amount of any other direct or indirect damage suffered by the customer.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adaptation to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. ANY IMPLIED WARRANTIES, OBLIGATIONS, OR LIABILITIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTY OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED IN DURATION TO THE DURATION OF THIS APPLICABLE WRITTEN WARRANTY. Some states do not allow limitations on how long an implied warranty lasts, so the above limitations may not apply to you.

IN NO EVENT SHALL BEURER BE LIABLE FOR ANY SPECIAL, INCIDENTAL, INDIRECT OR CONSEQUENTIAL DAMAGES FOR BREACH OF THIS OR ANY OTHER WARRANTY, EXPRESS, IMPLIED OR ANY OTHER THEORY OF LIABILITY, WHATSOEVER. Some states do not allow the exclusion or limitation of special, incidental, or consequential damages, so the above limitation may not apply to you.

Beurer does not authorize anyone, including, but not limited to, retailers, the subsequent consumer purchaser of the product from a retailer or remote purchaser, to obligate Beurer in any way beyond the terms set forth herein.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on Internet auction sites and/or products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate in connection with any products or parts thereof which are repaired, replaced, altered, or modified, without the prior explicitly written consent of Beurer.

This warranty gives you specific legal rights, and you may also have other rights which may vary from state to state.

For more information regarding our product line in the USA, please visit: www.beurer.com

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